

# BESTSLING SPECIFICATIONS

## MATERIAL CARE SIZING

Material plays a big part in the comfort of any type of transfer. Bestcare provides many different types of durable mesh, disposable, and full fabric material.



**BESTCARE BLUE  
POLYESTER**



**MESH BLUE  
FOR BATHING APPLICATIONS**  
-loosely woven fabric that has a large number of closely spaced holes



**HEAVY DUTY GRADE  
PERIWINKLE**  
-polyester  
-acrylic coated



**SINGLE PATIENT SPECIFIC  
NONWOVEN DISPOSABLE**  
-infection control  
-polypropylene  
-latex free



**SPECIAL WOVEN DISPOSABLE  
PERIWINKLE**  
-polyester  
-acrylic coated fabric

# CARE & WASHING

Care instructions on the sling label shall have precedence over these instructions should the two be in disagreement. Always follow the care instructions on the sling label. Never wash a Disposable or Single Patient Specific (SPS) sling. Never use bleach to wash or clean a Bestcare sling as the material may be damaged. For reusable fabric slings, hand or machine wash in a gentle cycle with mild soap solution at a maximum temperature of 82°C / 180°F. Rinse thoroughly and air dry. Do not dry clean and always wash slings separately from other items.


## CAUTION!



**DO NOT USE BLEACH WITH ANY SLING. THE SLING WILL BE DAMAGED AND UNSAFE TO USE!**

**DO NOT EXCEED WEIGHT CAPACITY.**

# SIZE & BODY WEIGHT

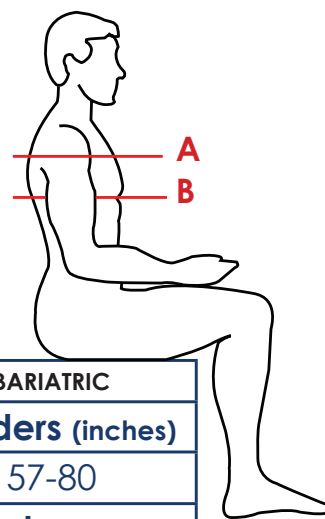


	SMALL	MEDIUM	LARGE	BARIATRIC
<b>A - diameter around the shoulders (inches)</b>				
UNIVERSAL SPLIT LEG	28-45	37-57	47-67	57-80
<b>B - diameter around the chest (inches)</b>				
PEDIATRIC	18-28	N/A	N/A	N/A
SANI/TOILETING	18-40	26-46	28-50	34-73
SIT-TO-STAND	26-42	30-46	34-53	46-64
<b>body weight (LB)</b>				
UNIVERSAL SPLIT LEG	85-140	140-200	200-400	400-600
	MEDIUM	LARGE	X-LARGE	XX-LARGE
R-SERIES	100-195	150-300	210-400	290-600

*\*above dimensions are guidelines and may vary by user*




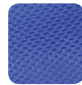
# UNIVERSAL SLING


Universal slings or “U” slings allow for easier application than full body slings. The universal sling molds around the body easily, eliminating the need to pull the sling under and risk skin injury or discomfort. Available in mesh or full deluxe padded styles. Six point connectivity increases safety and weight distribution for superior comfort. Also available with full head support.

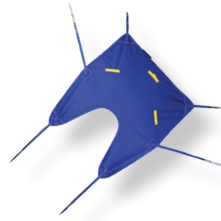


## UNIVERSAL SLINGS without head support


material type	part number	description	size	best fits	weight capacity
 bestcare blue	SL-UP892	padded	M	140-200LB	400LB
	SL-UP893	padded	L	200-400LB	400LB
	SL-UP894	padded	XXL	400-600LB	600ILB

material type	part number	description	size	best fits	weight capacity
 mesh	SL-UM882	mesh	M	140-200LB	400LB
	SL-UM883	mesh	L	200-400LB	400LB

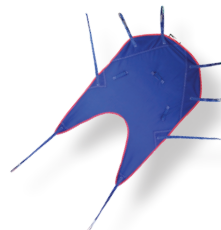

## UNIVERSAL SLINGS with head support


material type	part number	description	size	best fits	weight capacity
 bestcare blue	SL-UPH861	padded	S	85-140LB	400LB
	SL-UPH862	padded	M	140-200LB	400LB
	SL-UPH863	padded	L	200-400LB	400lb
	SL-UPH864	padded	XXL	600LB	600LB


material type	part number	description	size	best fits	weight capacity
 mesh	SL-UMH831	mesh	S	85-140LB	400LB
	SL-UMH832	mesh	M	140-200LB	400LB
	SL-UMH833	mesh	L	200-400LB	400ILB

BEST SLING

# SANI-SLING

Toileting sling made for maximum functionality and patient dignity. Eliminates need for poorly designed commode opening slings. Easy to apply and use, these slings are well padded under the arms and lumbar region for maximum comfort. Bestcare Sani slings also work extremely well on stand assist lifts for patients requiring more lower back support during lifts.

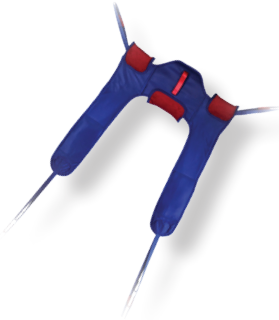

material type	part number	description	size	torso	weight capacity
 bestcare blue	SL-TP331	padded	S	26"-42"	400LB
	SL-TP332	padded	M	30"-48"	400LB
	SL-TP333	padded	L	38"-60"	400LB
	SL-TP334	padded	XXL	58"-73"	600LB

# STANDASSIST

Experience the most comfortable stand assist sling on the market! Lumbar and under-arm pads provide maximum comfort for the patient. For those that require some additional initial lift assistance optional buttock strap attachment is available and a calf strap is also available for stabilization support.

SPS stand assist



stand assist





buttock strap




calf strap




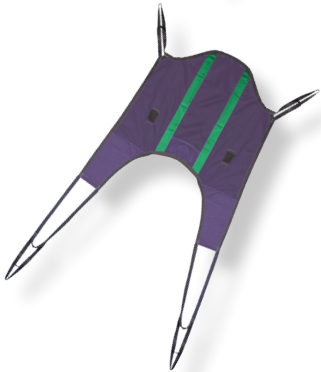
material type	part number	description	size	torso	weight capacity	unit
 disposable	SL-DSA662	SPS disposable stand assist padded (10/box)	M	29"-48"	400LB	10/cs
	SL-DSA663	SPS disposable stand assist padded (10/box)	L	36"-60"	600LB	10/cs
 bestcare blue	SL-SA661	deluxe padded (toileting)	S		400LB	
	SL-SA662	deluxe padded (toileting)	M		400LB	
	SL-SA663	deluxe padded (toileting)	L		400LB	
	SL-SA664	deluxe padded (toileting)	XXL		600LB	
	SL-SA669	buttock support strap for stand assist standard	standard		400LB	
	SL-SA669B	buttock support strap for stand assist standard	XXL		600LB	
	SAL1230-KB	stand assist knee strap brace	standard		400LB - 600LB	

# REPLACEMENT SLINGS

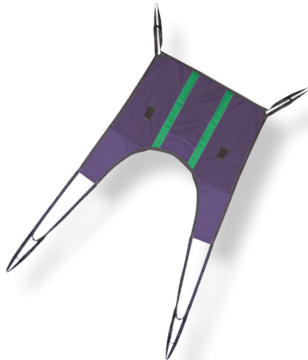


## REPLACEMENT for LIKO®/GULDMANN® without head support

material type	part number	description	size	weight capacity
 periwinkle	SL-GP471	low back	S	600LB
	SL-GP472	low back	M	600LB
	SL-GP473	low back	L	600LB
	SL-GP474	low back	XL	600LB




## REPLACEMENT for LIKO®/GULDMANN® with head support

material type	part number	description	size	weight capacity
 periwinkle	SL-GPH481	high back	S	600LB
	SL-GPH482	high back	M	600LB
	SL-GPH483	high back	L	600LB
	SL-GPH484	high back	XL	600LB



## REPLACEMENT for HOYER® without head support

material type	part number	description	size	best fits	weight capacity
 bestcare blue	SL-HC70002	padded	M	99-210lb	500LB
	SL-HC70001	padded	L	198-350lb	500LB
	SL-HC70000	padded	XL	270-600lb	600LB




## REPLACEMENT for HOYER® with head support

material type	part number	description	size	best fits	weight capacity
 bestcare blue	SL-HC70012	padded	M	270-600lb	600LB
	SL-HC70011	padded	L	270-600lb	600LB
	SL-HC70010	padded	XL	270-600lb	600LB



## REPLACEMENT for INVACARE®


full body padded without commode opening

material type	part number	description	size	best fits	weight capacity
 bestcare blue	SL-R112	full body	M	100-195lb	450LB
	SL-R113	full body	L	150-300lb	450LB
	SL-R117	full body	XL	200-400lb	450LB



## REPLACEMENT for INVACARE®


full body mesh without commode opening

material type	part number	description	size	best fits	weight capacity
 mesh	SL-R110	full body	M	100-195lb	450LB
	SL-R111	full body	L	150-300lb	450LB
	SL-R140	full body	XXL	290-600lb	600LB



## REPLACEMENT for INVACARE®


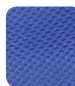
full body with commode opening

material type	part number	description	size	weight capacity
 mesh	SL-R114	with commode opening	M	450LB
	SL-R115	with commode opening	L	450LB
	SL-R116	with commode opening	XL	450LB
	SL-R141	with commode opening	XXL	600LB




## PEDIATRIC

deluxe padded

material type	part number	description	size	weight capacity
 bestcare blue	SL-PP121	padded	small	100LB or less
 mesh	SL-PM111	mesh	small	100LB or less

# DISPOSABLE SLINGS


Bestcare Single Patient Slings are made from a polypropylene nonwoven layered fabric which allows for a cost effective method of controlling cross contamination and serves as an excellent alternative to woven cloth slings whose cost accumulates over time along with required upkeep and laundering. When more slings are available, staff can assign a sling to each individual Patient. No more sharing slings. No chasing down slings that have gone missing with the laundry. Latex free and tested at 1.5 times the safe working load. SPS slings are intended to be used with a single patient only and cannot be laundered. When soiled, dispose of the sling accordingly.

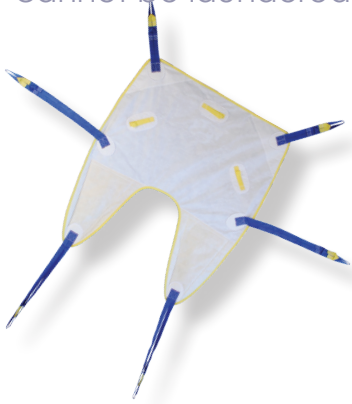

## UNIVERSAL SPS

universal sling without head support

material type	part number	description	size	weight capacity	unit
disposable	SL-UD812	single patient specific	M	600LB	10/cs
	SL-UD813	single patient specific	L	600LB	10/cs

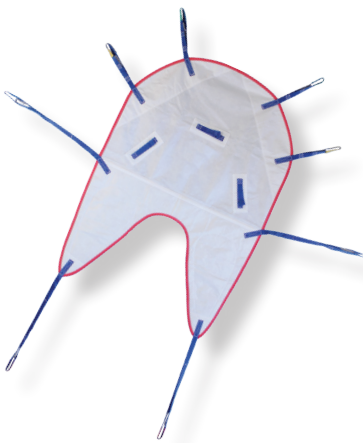

## UNIVERSAL SPS

universal sling with head support


material type	part number	description	size	weight capacity	unit
disposable	SL-UDH821	single patient specific	S	600LB	10/cs
	SL-UDH822	single patient specific	M	600LB	10/cs
	SL-UDH823	single patient specific	L	600LB	10/cs
	SL-UDH824	single patient specific	XL	600LB	10/cs



## STAND ASSIST SPS


velcro belt for better fit and provides patient with more sense of safety and security


material type	part number	description	size	weight capacity	unit
disposable	SL-DSA662	single patient specific	M	400LB	10/cs
	SL-DSA663	single patient specific	L	400LB	10/cs

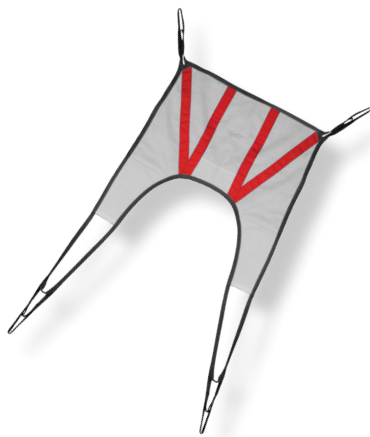

## REPLACEMENT for LIKO®/GULDMANN® SPS


disposable full body sling with out head support


material type	part number	description	size	weight capacity	unit
 SPS	SL-GD431	SPS basic low back	S	600LB	10/cs
	SL-GD432	SPS basic low back	M	600LB	10/cs
	SL-GD433	SPS basic low back	L	600LB	10/cs
	SL-GD434	SPS basic low back	XL	600LB	10/cs

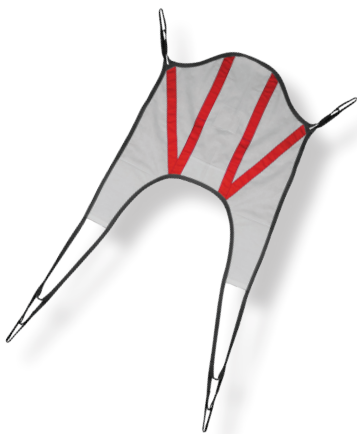

## REPLACEMENT for LIKO®/GULDMANN® SPS

disposable full body sling with head support


material type	part number	description	size	weight capacity	unit
 disposable	SL-GDH441	SPS high back	S	600LB	10/cs
	SL-GDH442	SPS high back	M	600LB	10/cs
	SL-GDH443	SPS high back	L	600LB	10/cs
	SL-GDH444	SPS high backK	XL	600LB	10/cs

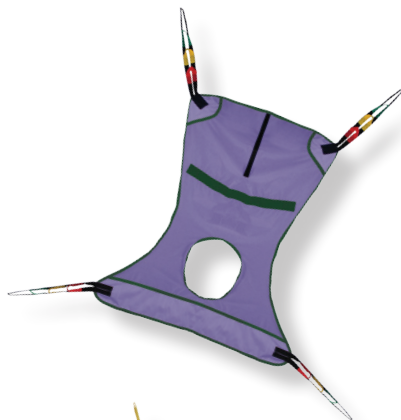

## REPLACEMENT for INVACARE® SPS


disposable full body with commode opening

material type	part number	description	size	best fits	weight capacity	unit
 special woven	SL-RD114	disposable	M	100-195lb	600LB	10/cs
	SL-RD115	disposable	L	150-300lb	600LB	10/cs
	SL-RD116	disposable	XL	210-400lb	600LB	10/cs
	SL-RD141	disposable	XXL	290-600lb	600LB	10/cs

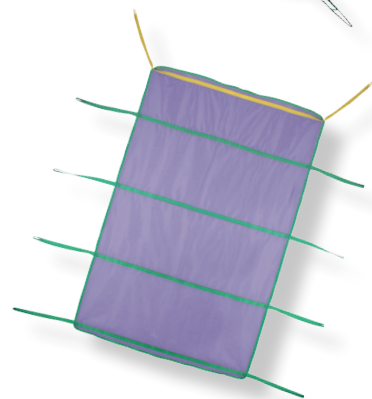

## REPOSITIONING SLING SPS

disposable repositioning sling for single patient use

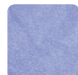material type	part number	description	size	weight capacity	unit
 special woven	SL-RPD1000	disposable	standard	1000LB	10/cs
	*1000lb safe work load is ONLY when all loop straps are properly attached				